

Cartridge handle: Pulling this handle activates HeartStart and begins voice instructions. **Ready light:** Blinks to indicate that HeartStart is ready for use.

On/Off button: Pressing this button activates HeartStart and begins voice instructions.

SMART pads cartridge: Contains two adhesive pads to be placed on the patients chest in an emergency.


Information button (i-button): Push the "i" button when it is flashing to hear additional instructions or information, for example instructions on cardio pulmonary resuscitation.

Caution light: This light flashes during HeartStarts' heart rhythm analysis and is on when a shock is advised as a reminder that no one should touch the patient.

Shock button: Flashes when HeartStart determines that a shock is necessary. HeartStarts' voice instructions will tell you when to press this button.

If someone collapses and is UNRESPONSIVE and NOT BREATHING PROPERLY you must first call the emergency services (999). If possible delegate someone else to call the emergency services.

Get your Philips HeartStart Defibrillator and follow these steps:

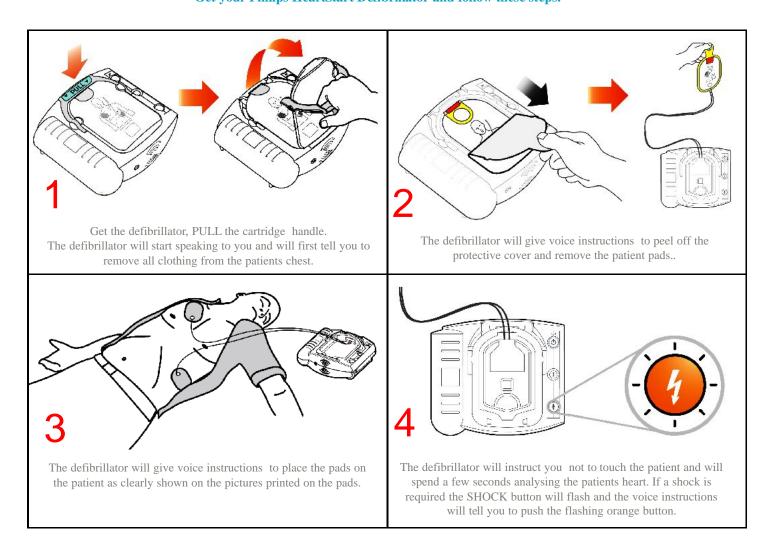

The defibrillator will continue to monitor the patients heart and will give voice instructions telling you what actions to take next. If additional shocks are required it will advise you to press the orange flashing SHOCK button. If cardio pulmonary resuscitation (CPR) is required it will provide voice instructions on how to give effective CPR. The defibrillator should remain attached to the patient until the emergency services arrive.